DEUTSCH

SanoSkin®- Melladerm® Plus - Tulle 10 x 10 cm

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English**SanoSkin®- Melladerm® Plus -Tulle 10 x 10 cm**

SanoSkin® Melladerm® Plus-Tulle is a fatty gauze made of specially woven, wide gauge, polyester mesh impregnated with SanoSkin®- Melladerm® Plus honey gel. The gauge of the mesh was selected to allow wound fluid to pass through easily so it can be absorbed by the secondary dressing (compress or foam). The SanoSkin®-Melladerm® Plus honey gel consists of; Honey, PEG 4000, Propylene glycol, and glycerol.

When the Melladerm® Plus -Tulle is in contact with the wound, fluid is extracted from the surrounding tissues. A moist wound healing environment together with the honey stimulates the wound healing process, resulting in less inflammation (less swelling and pain), facilitating autolysis and promoting better epithelial cell migration.

Indications

SanoSkin®- Melladerm® Plus -TULLE is used for the treatment of superficial acute and chronic wounds.

Contraindications

Do not use if package is damaged or opened. No contraindications are known to date. Do not use in cases where the patient is very susceptible to infection e.g. in large burns. Do not use on individuals with a known sensitivity to the Melladerm® Plus gel or its components.

Precautions and observations

The presence of bacteria in wounds is normal and is not a contra indication.

When an infection is suspected during the use of SanoSkin® Plus -Tulle the correct antibiotic therapy should be started by a physician.

Depending on the underlying pathology, wound care must be used in conjunction with correct medical support, treatment and supervision.

SanoSkin®- Melladerm® Plus -Tulle creates a moist healing environment which stimulates the autolytic debridement and the removal of necrotic tissue. As a result of this, it is possible that the wounds become bigger at the beginning of the treatment.

Instructions for use

- Clean wound gently with wound cleanser
- If needed, dry surrounding skin
- Remove support paper with sterile gloves or pincers and place SanoSkin®-Melladerm® Plus-Tulle on the wound.
- Never put several layers of TULLE over each other, the holes in the mesh could become clogged and wound fluid will no longer be able to drain away, which could result in maceration.
- SanoSkin®-Melladerm® Plus-Tulle can be cut with sterile scissors into the required size. If necessary extra SanoSkin®-Melladerm® PLUS gel can be applied from a tube.
- SanoSkin®-Melladerm® Plus-Tulle can be covered with a secondary dressing and fixed with a bandage or tape.

Removal

The frequency of dressing changes would depend on the condition of the wound, but is usually every two days.

Storage precautions

Store at room temperature, maximum 25° Celsius / 77° F

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