NEDERLANDS

SANOSKIN Melladerm® PLUS

10 - 20 – 30 - 50 gram tubes

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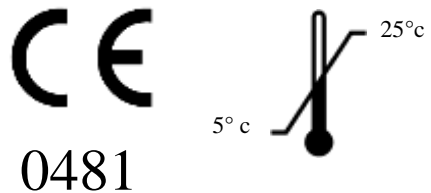
10 - 20 - 30 - 50 gram tubes

Product description

Melladerm® PLUS is a gel that contains: honey, glycerine, propylene glycol, PEG 4000. When the gel is in contact with the wound, fluid is extracted from the surrounding tissues. Due to the osmotic action a moist wound-healing environment is created. A moist wound healing environment together with the honey stimulates the wound healing process, facilitates autolysis and promotes better epithelial cell migration. The gel will dilute gradually and can be removed easily with a wound cleanser. *Melladerm® PLUS* is a primary wound dressing that can be covered by most commonly used secondary dressings.

Indications

Superficial Chronic wounds - Decubitus, Ulcus Cruris, diabetic foot wounds, Fungating wounds (deodorising and debriding).
Contaminated Acute wounds – surgical wounds (postoperative wounds), Traumatic wounds (superficial wounds, cuts), Small burn and laser wounds (1° and 2° degree).

Contraindications

Do not use if package is damaged or opened. No contraindications are known to date. Do not use the gel in cases where the patient is very susceptible to an infection e.g. in very large burn wounds. Do not use on individuals with a known sensitivity to the gel or its components.

Instructions for use

1. Remove lid, and remove aluminium seal in an aseptic way.
2. Squeeze the tube and fill the wound, taking care not to touch the wound.
3. Use a (sterile) spatula when necessary.
4. Place a dressing on top of the gel to cover the wound.
5. *Melladerm® PLUS* can stay on the wound up to 48 hours on average. Renew the gel in the wound as soon as the gel has disappeared.

Precautions and observations

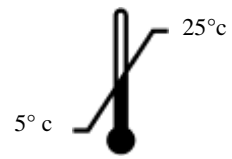
- * Consult a healthcare professional for the appropriate medical treatment when appropriate.
- * Inspect and clean the wound with a wound cleanser.
- * Bacterial colonisation of chronic wounds is always present and is not a contraindication for using *Melladerm® PLUS*.
- * Due to the autolytic debridement the wound may appear deeper after the first dressing changes.
- * Seek medical supervision if signs of infection occur.
- * *Melladerm® PLUS* is for a single patient use only.
- * When the condition of the wound deteriorates, treatment with *Melladerm® PLUS* should be stopped.
- * After the packaging has been opened and the product is used in accordance with the instructions described in the leaflet, the storage life of the product is equal to the expiry date printed on the packaging.

Storage precautions

Store at room temperature, maximum 25° Celsius

Manufactured/fabriqué by/par
SanoMed Manufacturing B.V.
Smoutweg 4 -008
4524 EK Sluis
The Netherlands
E- Mail: Info@SanoMed.nl
www.sanomed-manufacturing.eu

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© SanoMed Manufacturing BV 2008 Barring Spelling errors, SanoMed Manufacturing BV shall at all times retain the right to change its products and assortment.