NEDERLANDS

Melloxy®
15 - 20 – 30 - 50 gram tubes

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SanoSkin®–Melloxy®
15 - 20 – 30 - 50 gram tubes

Product description

Melloxy® is a gel that contains: honey, ozonated vegetable oil, propylene glycol, PEG 4000. When the gel is in contact with the wound, fluid is extracted from the surrounding tissues through osmosis. By doing so a moist wound-healing environment is created. This environment stimulates the wound healing process, facilitates autolysis and promotes better epithelial cell migration. Due to the strong osmotic activity *Melloxy®* works anti-microbial. The gel will dilute gradually and can be removed easily with a wound cleanser. *Melloxy®* is a primary wound dressing that can be covered by most commonly used secondary dressings. The gel may contain sugar crystals, which will however not affect the effectiveness and/or application of the gel. Due to the use of natural products each batch of gel may have a slightly different colour (white - yellow - brown).

Indications

Chronic wounds - Decubitus (Stadium I – IV), Ulcus Cruris, diabetic foot wounds, Fungating wounds (deodorising and debriding).
Contaminated Acute wounds – surgical wounds (postoperative wounds), Traumatic wounds (superficial wounds, cuts), Small burn and laser wounds (1^o and 2^o degree).

Contraindications

No contraindications are known to date. Do not use on individuals with a known sensitivity to the gel or its components. Do not use if package is damaged or opened. Do not use the gel in cases where the patient is very susceptible to an infection e.g. in very large burn wounds.

Precautions and observations

1. Consult a healthcare professional for the appropriate medical treatment.
2. Inspect and clean the wound. Appropriate supportive measures should be taken where indicated.
3. Due to the autolytic debridement the wound may appear deeper after the first dressing changes.
4. Bacterial colonisation of chronic wounds is always present and is not a contraindication for using *Melloxy®*
5. Seek medical supervision if signs of infection occur (fever, pain, redness, and pus).
6. Treatment with *Melloxy®* should be stopped if the condition of the wound deteriorates.
7. *Melloxy®* is for a single patient use only.
8. After the packaging has been opened and the product is used in accordance with the instructions described in the leaflet, the storage life of the product is equal to the expiry date printed on the packaging.

Instructions for use

1.Remove lid, clean the mouth of the tube and the top of the lid with an alcohol swab.
2. Push the top of the lid into the silver coating to break the seal
3. Squeeze the tube and fill the wound, taking care not to touch the wound.
4.Use a (sterile) spatula when necessary.
5. Place a dressing on top of the gel to cover the wound.
6. *Melloxy®* can stay on the wound up to 48 hours on average. Renew the gel in the wound as soon as the gel has disappeared.

Storage precautions

Store at room temperature, maximum 25° Celsius / 77° F

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© SanoMed Manufacturing BV 2006 Barring Spelling errors, SanoMed Manufacturing BV shall at all times retain the right to change its products and assortment.