Nederlands

SanoSkin®-Net
Ref SN1285

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SanoSkin®-Net
Ref SN1285

Product description:

SanoSkin®-Net is a hydrogel coated dressing on an open weave polyester net.

The dressing contains water and acrylic polymers. The wound exudate will be held in the gel thus providing a moist and optimal wound healing environment. This environment creates an anti-inflammatory response, lessens the swelling and pain, and cleanse the wound resulting in optimal wound healing.

The dressings will not adhere to the wet wound and upon removal will prevent damage to the new granulating tissue and the surrounding intact skin. SanoSkin®- Net is to be used in conjunction with a secondary dressing (e.g. absorbing dressing/gauze compress). SanoSkin®- Net can be combined with SanoSkin®-Gel.

Indications:

SanoSkin®- Net is a multipurpose wound contact layer providing some exudate absorption and retention. The dressings are indicated for acute and chronic superficial wounds such as: bruises, skin tears, pressure ulcers, venous and arterial ulcera, diabetes ulcers, donor sites, cuts, 1st and 2nd degree burns, postoperative wounds and other external wounds caused by trauma. SanoSkin®-Net is to be used in conjunction with other secondary dressings. The dressing can stick to the dry surrounding skin and to remove the dressing without pain, use some saline.

Contra-indications:

It should not be used on its own as a covering for dirty infected and heavily exudating wounds (should first be cleansed), third degree burns, deep narrow cavities.

Instructions for Use:

- Consult a healthcare professional for the appropriate medical treatment.
- Select a dressing that will cover the wound and extend onto healthy tissue. (Approximately 25mm)
- Nettoyez la plaie si nécessaire .
- Remove the sterile dressing from the package.
- To apply the dressing remove the dear polyester liner.
- Position and gently place the dressing, with the gel impregnated side facing downward, on the wound.
- Place a secondary dressing and secure.
- Inspect the wound and the surrounding tissue regularly.

* Do not use if packaging damaged or compromised in any way.

* Do not use after expiry date.

* In case of an intolerance to the dressing or one of its components, remove the dressing and clean the area carefully.

Frequency of Change:

SanoSkin®-Net can stay on the wound several days (3-5 days). It should be changed as often as the condition of the wound dictates. When changing the secondary dressing the Net can stay on the wound. When the gel sticks to the skin it can easily be removed by using some saline.

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