NEDERLANDS

SanoSkin®-OXY

30 gram tubes

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30 gram tubes

Product description

SanoSkin®-OXY is a fatty ointment that consists of ozonated vegetable oil. When the ointment comes into contact with wound fluid/water, the ozonides gets released and spreads the typical odour.

Indications

SanoSkin®-OXY is used in the treatment of superficial wounds. The gel creates a moist wound healing environment that helps the autolytic debridement of necrotic tissue.

Contra-indications

Patients who are known to be sensitive to the ointment or one of its components.

Precautions

The presence of bacteria in wounds is a normal phenomenon and is no contra indication. If an infection is suspected or is developing during the use then a correct antibiotic therapy should be started. Depending on the underlying pathology the wound care treatment should always be accompanied by the correct medical treatment and supervision. SanoSkin®-OXY creates a moist wound climate which stimulates the debridement of necrotic tissue. Due to this it is normal that the wound surface becomes bigger.

Application

1. Clean the wound.
2. Open tube in an antiseptic way.
3. Bring ointment directly on/in the wound and make sure the ointment does not come above the wound margins.
4. Cover the ointment/wound with a moisture retention dressing.

Remove

SanoSkin®-OXY should be changed when leakage of the secondary dressing occurs or when the secondary dressing needs to be removed. The ointment can be cleansed out of the wound with a sterile saline solution. Dressing change is depending on the wound fluid production of the wound, but is usual between 2 - 7 days.

Storage precautions

Store between 5°C and 25°C, prevent freezing

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© SanoMed Manufacturing BV 2006 Barring Spelling errors,